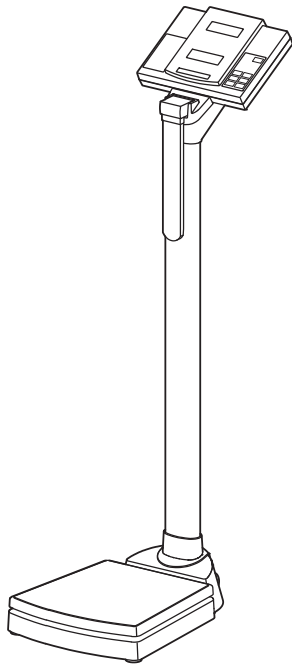
seca 764

D	<i>Bedienungsanleitung und Garantieerklärung</i>	3
GB	<i>Instruction manual and guarantee</i>	17
F	<i>Mode d'emploi et garantie</i>	31
I	<i>Manuale di istruzioni e garanzia</i>	45
E	<i>Manual de instrucciones y garantía</i>	59
DK	<i>Betjeningsvejledning og garantibevis</i>	73
S	<i>Bruksanvisning och garanti</i>	87
N	<i>Bruksanvisning og garantierklæring</i>	101
FIN	<i>Käyttöohje ja takuu</i>	115
NL	<i>Bedieningshandleiding en garantieverklaring</i>	129
P	<i>Instruções de utilização e declaração de garantia</i>	143
GR	<i>Οδηγίες χειρισμού και εγγύηση</i>	157
PL	<i>Instrukcja obsługi i gwarancja</i>	171

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Full certification

seca products you are purchasing not only technology matured over a century, but also quality confirmed by authorities, the legal system and by institutes.

seca products comply with European directives, standards and national laws. With seca, you are buying the future.

The products in these operating instructions comply with German legislation governing medical products and therefore with Directive 93/42/EEC of the Council of the European Community which is embodied in national laws throughout Europe.

Scales carrying this symbol are approved for medical purposes in the European Community. seca scales with this symbol meet the stringent quality and technical requirements placed on scales which can be calibrated.

Scales carrying this symbol are calibrated in accordance with Precision Class III of EC Directive 2009/23/EC.

Products carrying this symbol comply with the following directives and standards:

1. Directive 2009/23/EC governing non-automatic weighing instruments
2. Directives 93/42/EEC and 2007/47/EC on Medical Devices
3. DIN EN 45501 governing metrological aspects of non-automatic scales

seca's professionalism has also been officially recognized. The certificate from TÜV Product Service, the body responsible for medical devices, confirms that as a manufacturer of medical devices, seca rigorously complies with stringent legal requirements. seca's quality assurance system covers the design, development, production, sales and service of medical scales and measuring systems.

seca helps the environment.

We are concerned to save natural resources. We therefore strive to save packaging material where meaningful. What is left over can conveniently be disposed of locally via the German Dual System (recycling programme).

1. Congratulations!

Your purchase of this integrated measurement station **seca 764** means you have a high-precision and sturdy electronic body height and body weight measuring instrument. seca has been putting its experience at the service of health for over 150 years now, and as market leader in many countries of the world, is always setting new standards with its innovative developments for weighing and measuring.

The integrated **seca 764** measurement station is primarily used in accordance with national regulations in hospitals, doctor's practices and health care centers.

The measurement station is a medical product falling under Class I and is calibrated to meet Precision Class III. The integration of electronic scales and electronic height measurement in one

device means the **seca 764** is particularly suitable where weighing and measuring must be carried out where little space is available. The data measured can be converted at the push of a button into a Body Mass Index. The telescoping design used for the height measurement unit as well as the retractable head slide allows the device to be easily moved on the built-in castors.

The sturdy design and intelligent power management system ensure a long period of maintenance-free service so that the device can be used for years to come. Operation is easy with two large displays allowing weight and height to be read out simultaneously.

2. Safety

Before using the device, please take some time to read the following safety instructions.

- Follow the safety instructions in the instruction manual.
- Keep the operating manual and the declaration of conformity contained therein in a safe place.
- Never drop the device or expose it to strong shocks.
- Have repairs carried out exclusively by authorised persons.
- After moving the device, place it carefully back in the position for use.
- After using the height measurement unit fold the head slide down. This avoids the hazard of injury.
- Be sure to use only the seca AC adaptor supplied. Before plugging the AC adaptor into a socket, check that the mains voltage indicated on the AC adaptor matches local mains voltage.
- When using the device with an AC adaptor, be sure that the power supply cable does not pose any danger of tripping or entanglement.
- Be sure to have the device maintained and recalibrated regularly (see "Maintenance").

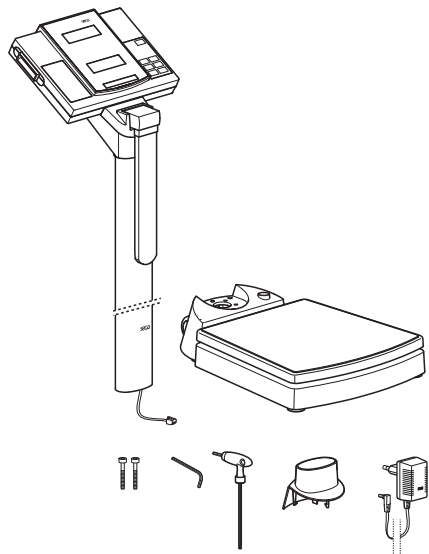
3. Before you start...

Unpacking

Remove the packaging and place the based on a solid, even surface.


The equipment supplied includes:

- Base of scales
- Column with integrated height measurement and display head
- Protective cover for column
- SW 3 Allen spanner
- SW 5 Allen spanner
- 2 screws
- seca AC adaptor
- Operating instructions with a declaration of conformity

Mounting the column

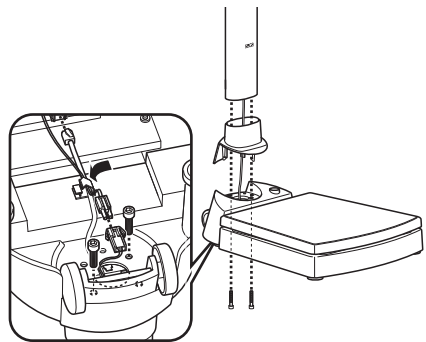- Place the protective cover onto the column.
- Place the column in to the notch so that the folding head slide is pointing down toward the platform.
- Screw the column from beneath onto the weighing platform.
- If needed, carefully place the scales on the side to tighten the screws.
- As shown in the illustration, connect the power supply cable on the bottom of the weighing platform.
- Insert the plug of the display cable in the jack on the bottom of the weighing platform.
- Fasten the cable in the cable guide.
- Now put the device back in an upright position.
- Slide the column cover all the way down until it locks into place on the weighing platform.


Power supply

The power supply must come from the supplied AC adaptor.

The connector for the AC adaptor is located under the rod.

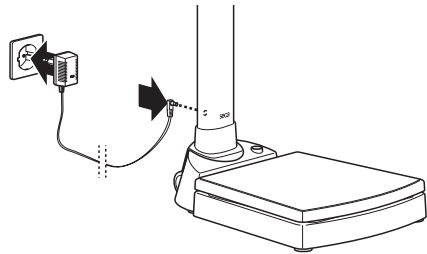Please use only the mains unit supplied or mains units from the seca range of accessories. Before plugging the unit into the socket, check whether the mains voltage marked on the mains unit matches local mains voltage.


4. Setting up and aligning the scales

- Place the scales on a level surface.
- Align the scales by screwing the foot screws. The air bubble in the spirit level must be right in the centre of the circle. The scales must stand securely on the floor on all four feet.

Important!

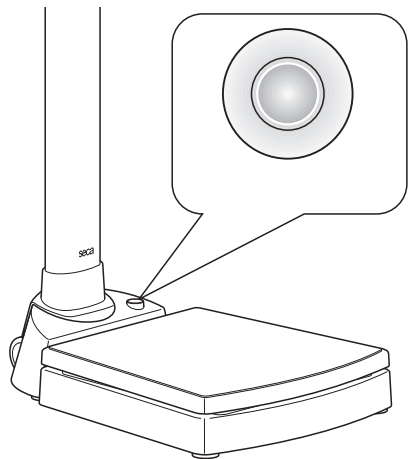
The alignment of the base of the scale must be checked and corrected if necessary every time the location of the scale is changed.

5. Operation

Controls and displays

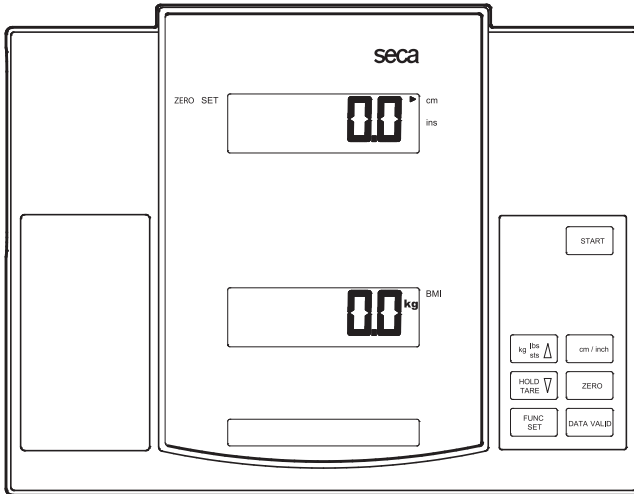The controls and display are in the display part.

- START

 Switching the device on and off

- kg lbs sts ▲

 Normal operation → No function
In the function menu → Raise the value

- HOLD TARE ▼

 Press button briefly → Active/deactivate hold function
Press button down → Set the display to zero, activate/deactivate TARE and hold briefly
In the function menu → Lower the value

- FUNC SET

 Activate BMI function, confirm

- cm / inch

 Toggling the unit of height measurement between cm and inch
This function is not available with all model variants

- ZERO

 Setting the height measurement to zero for relative measurements

- DATA VALID

 Pushing button to mark the valid height measurement value

Checking that the contents of the calibration counter are correct

These seca scales are calibrated. Calibration may be performed only by authorised centres. To ensure this, the scales are fitted with a calibration counter which records any change to data relevant to calibration. If you want to check whether the scales are properly calibrated, proceed as follows:

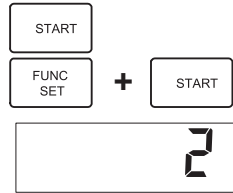
- If necessary, switch off the scales.
- Keep the **FUNC/SET** key depressed and start the scales.

The current contents of the calibration counter are shown on the display for a few seconds.

- Compare the output contents of the calibration counter with the number shown on the calibration counter sticker.

For valid calibration, both numbers must match. If sticker and calibration counter do not match, then re-calibration is required. Contact your service agent or seca Customer Service.

If a supplementary calibration is necessary, then the supplementary calibration counter sticker illustrated opposite is used to record the calibration counter position instead of the seca calibration sticker shown above. This sticker is secured with an additional seal by the person authorised to perform the supplementary calibration. The supplementary calibration mark is available from seca Customer Service under the number 14-05-01-886.

Both numbers must match.

Correct weighing

- Press the Start key with no load on the scale.

SECA, **88888** and **00** appear consecutively in the display. The scale is then automatically set to zero and ready to use.

- Stand on the scales and remain motionless.
- Read the weighing result off the digital display.
- If a weight of more than 250 kg is placed on the scales, then **STOP** appears in the display.
- To switch off the scales, press the Start button again.

Switching off the device

- To switch off the scales, press the Start button again.

Taring weight display (TARE)

The TARE function is useful if an additional weight should not be considered while weighing. Proceed as follows:

- Press the Start key with no load on the scale.
- First, place the additional weight onto the scales and activate the TARE function by pressing down and briefly holding down the **HOLD/TARE** button.

The display is reset to zero. **0.0** appears in the display. The note "NET" appears.

- Now place the patient onto the scales with the previously weighed additional weight.

The scales show the weight of the patient without the additional weight. You can now perform as many weighing operations as you like.

- Deactivate the TARE function by pressing the button down and holding it briefly once again; this puts the scales back into the normal weighing mode, the tare value (the additional weight value) is deleted.
- The tare value is likewise erased when the scales are switched off.

Store weight value (HOLD)

You can also continue displaying the weight determined once the load has been removed. This makes it possible to look after a patient first, before having to note the weight:

- Press the **HOLD/TARE** button while the scales have a weight on it.
- "HOLD" appears in the digital display. The weight remains stored in the display until the Hold function is automatically switched off.
- The triangle with the exclamation mark indicates that this frozen display is an additional facility which is not subject to calibration.
- The HOLD function can be switched off by pressing the **HOLD/TARE** key again. **0.0** appears in the display if the scales have no load, otherwise the current weight is shown. The "HOLD" display vanishes.

Note: The HOLD function can also be used if the TARE function has been executed beforehand.

Height measurements

– Press the **START** button on the display housing to switch on the device.

First ----- appears in the height display. The unit selected before switching off the device is active and is marked by an arrow.

– If required, switch the unit of height measurement ("Toggling unit of height measurement").

– Raise the head slide up and push the head slide up far enough for the person whose height is being measured to easily stand under it.

– Ask the person being measured to place his back against the height rod. Make sure the person's posture is upright and head is straight.

– Now push the head slide down until it rests on the person's head.

– Note down the height on the display.

Toggling unit of height measurement

Note:

This function is not available with all model variants.

The display can toggle between "cm" and "ins" (inches).

– Switch the device on. The height read-out is displayed in the most recent unit selected.

– To switch the unit displayed press the **cm/ins** button.

– An arrow lights up next to the **cm** or **ins** display.

Relative measurements

The height rod can be set to zero at any position. This function allows extremities to be measured.

- Press the **ZERO** button in order to set the height rod to zero at its current position.

ZERO SET is highlighted in the display. All lengths are now measured starting from this zero set point. Any measurement below this zero point will cause results to be displayed with a minus sign.

- Press the **ZERO** button again if you want to start measuring from absolute zero again.

Body Mass Index (BMI)

The Body Mass Index compares body size with body weight while also allowing more precise information to be determined for ideal body weight according to Broca. A tolerance range is given which is considered ideal for health.

By using the integrated measurement station **seca 764** you can have the Body Mass Index calculated directly from the size measured and the measured weight. Please follow these steps.

- Press the Start key with no load on the scale.

SECA, **88888** and **00** appear consecutively in the display. The scale is then automatically set to zero and ready to use.

- Now measure height as described. After you have determined that the patient is standing up straight and the head slide is correctly positioned, then please push the **DATA VALID** button. This marks the height measurement as a valid value for the Body Mass Index calculation.

- Push the head slide back up.
- Now read the weight value in the weight display.

- Press the **FUNC** key. The measured height which is marked as valid now appears in the display. Push the **kg/lbs/sts** buttons (raise the value) or **HOLD/TARE** (lower the value) if you want to correct the height, for example to compensate for shoe heel height.

- Push the **FUNC** button again The BMI function is now activated.
- Now the patient should try to stand as still as possible.
- Read the BMI on the LED and compare it with the categories under "Evaluating Body Mass Index".
- To return to normal weighing mode, press the **FUNC** key.
- To switch off the scales, press the Start button again.

Note:

The Body Mass Index is correctly calculated even when you have not select metric display units.

Evaluating Body Mass Index for adults:

Compare the value determined with the categories quoted below which correspond to those used by WHO, 2000.

- **BMI below 18.5:**

Warning, this patient weighs too little. There could be a tendency to anorexia. An increase in weight is recommended to improve well-being and performance. If in doubt, consult a specialist.

- **BMI between 18.5 and 24.9:**

The patient is of a normal weight. He can stay as he is.

- **BMI between 25 and 30 (pre-obese):**

The patient is slightly to moderately overweight. The patient should lose weight if a previous illness exists (such as diabetes, high blood pressure, gout, lipid disorders).

- **BMI over 30:**

Weight reduction is essential. Metabolism, cardiovascular system and bones are all under strain. A consistent diet, plenty of exercise and behavioural training are all recommended. If in doubt, consult a specialist.

6. Care and cleaning

When needed, clean the device with a household cleaning agent or other common disinfectant. Follow the manufacturer's instructions.

Under no circumstances use abrasive or acid cleaners, white spirit, benzene or the like for cleaning. Such substances can damage the high-quality surfaces.

7. What to do if ...

... no weight or height readout appears?

- Is the device switched on?
- Is the plug between the power supply connector in the rod and base properly connected?
- Is the plug contact of the AC adaptor properly connected on the device rod?
- Is the plug contact for connecting the display and the base properly connected?

... 0.0 does not appear before weighing?

- Press the start button again – there must not be any load on the scales – and only its feet should be in contact with the floor.

... one segment is illuminated constantly or not at all?

- The relevant segment has a fault. Inform Servicing.

... 5tOP appears in the weight display?

- Maximum load has been exceeded.

... the weight display is blinking?

- If you have just activated the Hold or Tare function, please wait until the scales have detected a steady weight value. The display will then stop flashing.
- If there has been no activation of a function beforehand, remove the load from the scales and wait until 0.0 is displayed, then weigh again.

... the display 111P appears?

- The ambient temperature of the device is too high or too low. Place the device where the ambient temperature is between +10 °C and +40 °C. Wait about 15 minutes until the device has adjusted to the ambient temperature and weigh again.

... the display E and a number appear?

- Use the Start button to switch off the device and start again. After this the device will function normally.
- If this is not the case, disconnect the power supply by briefly unplugging the AC adaptor. If this measure is equally unsuccessful, inform the Service department.

... ----- appears in the height display?

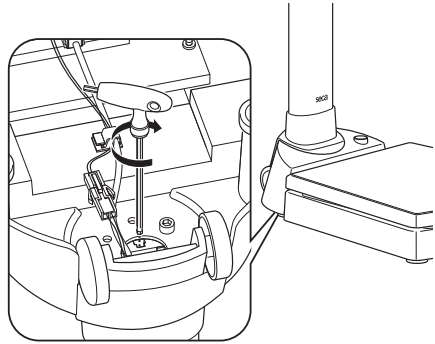
- Slide the head slide up or down quite a ways. This will make the head slide pass a mark which can be evaluated and the system can now calculate the position correctly again.

... the head stop slides down and does not stay in the required position?

- Place the scale carefully on its side so that the base of the scale is accessible. Insert the Allen key supplied in the rectangular hole on the end of the measuring rod as shown in the illustration and tighten up the screw for the measuring rod guide clockwise.

The measuring rod with the head stop should remain freely mobile, but not be able to slide down on its own.

Then stand the scale upright again.

8. Servicing/recalibration

Have the scales recalibrated by authorised persons in accordance with national legal requirements. The year of initial calibration can be found behind the CE symbol on the rating plate above the number of the appointed office, 0109 (Hessen Weights & Measures Office). Recalibration is required in all cases if one or more calibration seals have been damaged or calibration counter contents does not match the number on the valid calibration counter sticker (c.f. "Checking that the contents of the calibration counter are correct"). We recommend having your scales serviced by your local service agent before having it recalibrated. Seca Customer Service will be pleased to give you further assistance.

9. Technical data

Device dimensions

Width:	360 mm / 14.2 ins
Height:	1320 +10 mm / 52 +0.4 ins
Depth:	686 mm / 27 ins

Weight approx 20.5 kg

Temperature range +10 to +40 °C

Calibration as per Directive 2009/23/EC approved for medical use, Class III

Medicinal product according to 93/42/ECC Class I

Colour white/black

Maximum load 250 kg

Minimum load 2 kg

Graduations 100 g

Precision at first calibration

Up to 50 kg:	± 50 g
50 kg to 200 kg	± 100 g
200 kg to 250 kg	± 150 g

Height measurement

Precision Tolerance of ±2 mm

Reproducibility ±1 mm

Measurement range 110 - 200 cm / 43 - 79 ins

Resolution 1 mm / 0.05 ins

10. Spare parts

RS232 adapter kit with print port

Order no. 460-00-00-004

Model-specific power unit

Designation:

Plug-type AC adaptor 230 V~/50 Hz/12 V=/150 mA

Order no. 68-32-10-252

Designation:

Switchmode power adapter, 100-240V~ / 50-60Hz / 12V= / 0.5A

Order no. 68-32-10-265

Calibration seal

Order no. 14-05-01-886

11. Disposal

Disposing of the device

Do not dispose of the device in domestic waste. The device must be disposed of properly as electronic waste.

Follow the national regulations which apply in your case. For further information, contact our service department at:

service@seca.com

12. Warranty

A two-year warranty from date of delivery applies to defects attributable to poor materials or workmanship. All moveable parts - batteries, cables, AC adaptors, rechargeable batteries etc. - are excluded. Defects which come under warranty will be made good for the customer at no charge on production of the receipt. No further claims can be entertained. The costs of transport in both directions will be borne by the customer should the equipment be located anywhere other than the customer's premises. Warranty claims for

damage during transport can only be enforced if the complete original packaging was used in transit and the goods contained are secured and fixed into position in their original condition. All the packaging should therefore be retained.

A claim under warranty will not be honoured if the equipment is opened by persons not expressly authorised by seca to do so.

We would ask our customers abroad to contact their local sales agent in the event of a warranty matter.

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zero
set

cm
ins

The diagram shows a rectangular display area. On the left side, the text "zero" is above "set". On the right side, there is a right-pointing arrow above the text "cm", and below it, the text "ins". The display area itself contains seven horizontal dashes "-----".

zero
set

cm
ins

The diagram shows a rectangular display area. On the left side, the text "zero" is above "set". On the right side, there is a right-pointing arrow above the text "cm", and below it, the text "ins". The display area itself shows the number "173.5".

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