
# seca 874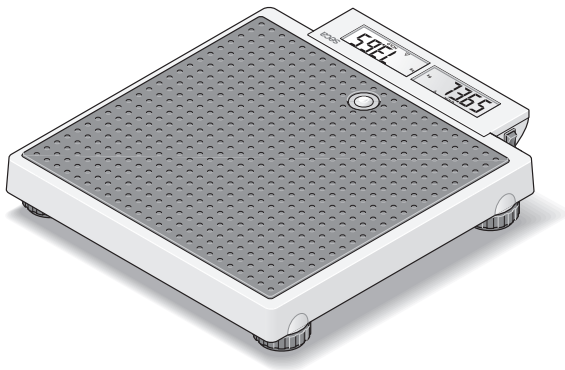

|           |   |           |
|-----------|---|-----------|
| <b>D</b>  | <b><i>Bedienungsanleitung und Garantieerklärung</i></b>         | <b>3</b>  |
| <b>GB</b> | <b><i>Instruction manual and guarantee</i></b>                  | <b>12</b> |
| <b>F</b>  | <b><i>Mode d'emploi et garantie</i></b>                         | <b>21</b> |
| <b>I</b>  | <b><i>Manuale di istruzioni e garanzia</i></b>                  | <b>30</b> |
| <b>E</b>  | <b><i>Manual de instrucciones y garantía</i></b>                | <b>39</b> |
| <b>P</b>  | <b><i>Instruções de utilização e declaração de garantia</i></b> | <b>48</b> |

# 1. Herzlichen Glückwunsch

---

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# 1. Congratulations

---

In the **seca 874** electronic personal scale you have acquired a highly-accurate and at the same time sturdy piece of equipment.

seca has been putting its experience at the service of health for over 170 years, and as market leader in many countries of the world, is always setting new standards with its innovative weighing and measuring developments.

In accordance with national regulations, the **seca 874** electronic personal scale is used primarily in hospitals, doctors' surgeries and mobile services. The scale is a

Class I medical device.

Weight display can be switched between kilogrammes (kg) and pounds (lbs). Some models of this scale **do not** have the unit-switching feature. Weight is displayed only in kilograms. Weight is determined within a few seconds.

A special feature of the **seca 874** scale is the "mother-and-baby" function. This enables the weight of babies and small children to be determined quite easily - in their mothers' arms.

The maximum load is 200 kg/440 lbs.

# 2. Safety

---

Before using the new scale, please take a little time to read the following safety instructions.

- Follow the instructions in the instructions for use.
- Use only the type of battery stated (see „Changing the batteries“ on page 18).
- When using the scale with a mains unit (option) or solar panel (option), make sure the arrangement of the power cable eliminates the risk of tripping or shunting.
- Do not drop the scale or subject it to violent shocks.
- Do not step on the display as this could damage it. Step only on the rubberized area.
- Have repairs carried out only by authorised persons.


# 3. Before you start ...

---

## Setting up the scale

The scale is fully assembled when dispatched.

- Remove the packaging and place the scale on a firm, level surface.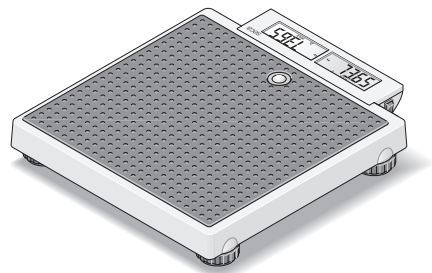

## Power supply

The scale is powered by batteries.

The LC display has a low power consumption - 10,000 weighing operations can be performed with one set of batteries. When the battery voltage is no longer adequate, **bAtE** appears in the display. You need 6 type AA 1.5 V batteries.


- Carefully turn over the scale so that the base is accessible.
- Press the closure of the battery compartment in the direction of the cover itself and open the battery compartment.
- Insert the supplied batteries into the battery compartment.

### Check that the polarity is correct.


- Close the cover and then turn the scale back up the right way.
- To activate the power supply, push the ON-OFF switch in position "ON".

#### Note

Some models of this scale are prepared for use with a seca mains unit or with a seca solar panel. These are available as optional accessories.

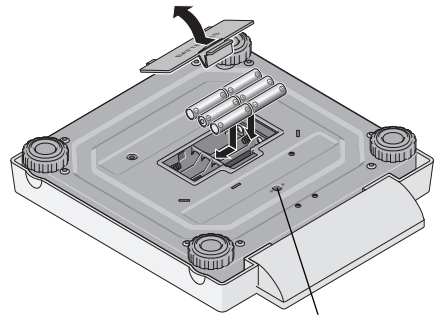
Connection (model-dependent) for mains unit/solar panel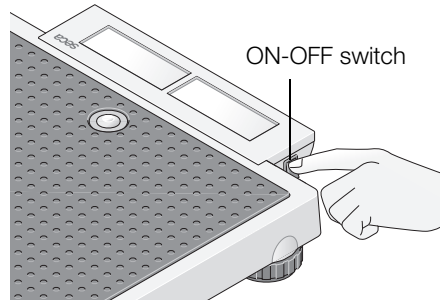




# 4. Operation

## Controls and displays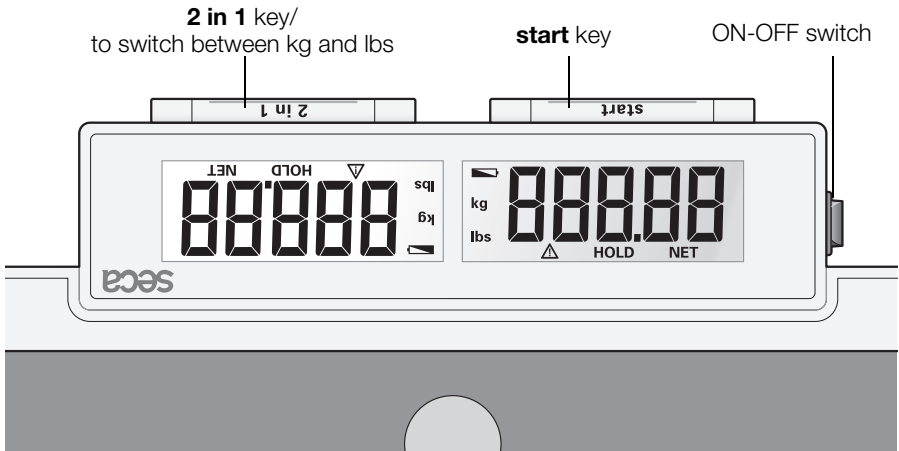

## How to weigh properly


- Press the **start** key with no load on the scale.  
*SECA*, *88888* and *0.00* appear consecutively in the display. The scale is then automatically set to zero and ready for use.
- If necessary, switch the weight display (see page 15).
- Ask the patient to mount the scale. Once on the scale, the patient should stand still.  
The HOLD function is activated automatically at weights of over 1.5 kg or 3.3 lbs. The display flashes until a stable weight has been measured. The display is then frozen until the next weighing operation.  $\Delta$  **HOLD** appears in the display.
- Read off the weighing result on the digital display.
- If a load greater than 200 kg or 440 lbs is placed on the scale, *STOP* appears in the display.
- To switch off the scale, press the **start** key again.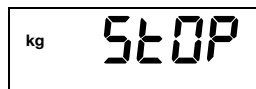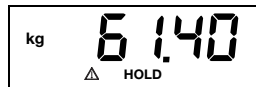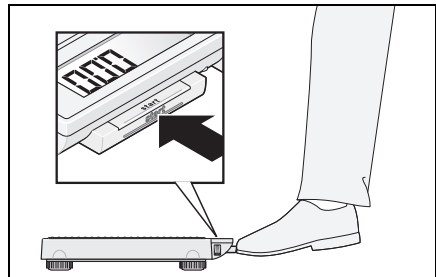


## Switching the weight display

The weight display can be switched between kilogrammes (kg) and pounds (lbs).

- Switch on the scale using the **start** key.  
Weight is initially displayed in the last setting selected.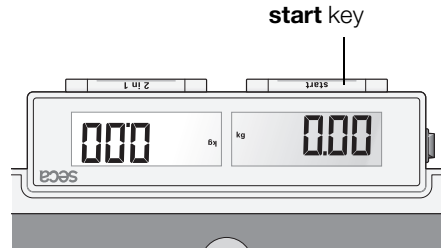


- To switch the weight display between kilogrammes and pounds, hold down the **2 in 1** key for approx. 3 seconds.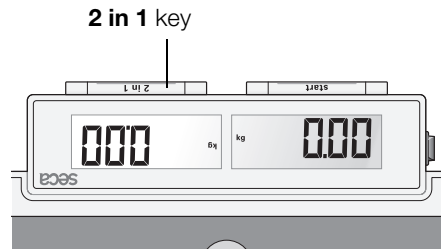

- The **kg** or **lbs** display lights up.

### Note

**Some models of this scale do not have the unit-switching feature. Weight is displayed only in kilograms.**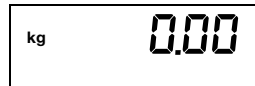

## Automatic switch-off/stand-by mode

If no further weighing operations are performed, the scale switches off automatically after a certain time:

in normal mode:

after 3 minutes

with the "2 in 1" function switched on:

after 2 minutes

## “2 in 1” function

The “2 in 1” function enables the body weight of babies and small children to be determined. The child is held in the arms of an adult. Proceed as follows.

- Press the **start** key with no load on the scale.
- Wait until the display **0.00** appears.
- Stand on the scale without the child. You can read off the weighing result on the digital display.
- Now press the **2 in 1** key to activate the function.

The scale stores the weight of the adult and the display returns to zero. **0.00** and the display **NET** appear in the display.

- Now take the child onto the scale. You can leave the scale to do this, in which case **-----** appears in the display.

The scale determines the weight of the child. Once the value is stable for about 3 seconds, the display is retained. This avoids the display jumping about as a result of the child’s movements.

△ **HOLD** and **NET** appear in the display.


**If several children are to be weighed consecutively, it is important that it is always the same adult who performs the measurement and that this person’s weight does not change (e.g. due to a piece of clothing being removed).**

The “2 in 1” function remains switched on until

- you press the **2 in 1** key again (total weight is displayed).
- the scale switches off automatically.

2 in 1 key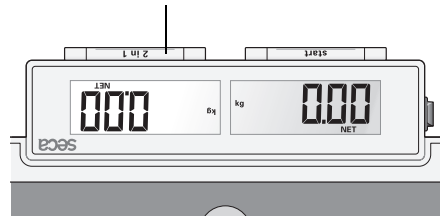

## 5. Cleaning

Clean the outside of the scale as required using a domestic cleaning agent or a commercially-available disinfectant. Follow the manufacturer’s instructions.

## 6. What do I do if ...

---

### ... no weight is displayed when there is a load on the scale?

- Is the scale switched on?
- Is the ON-OFF switch in position "ON"?
- Check the batteries.

### ... the scale keeps switching on, while being transported, for example?

- The **start** key has been pressed. Due to the very low energy requirement of the scale, the service life of the batteries will not be impaired. For longer journeys, you can push the ON-OFF switch in position "OFF" if necessary.

### ... the scale displays a weight after being transported or after new batteries have been put in?

- Press the Start key again. The scale will then work normally again.

### ... 0.00 does not appear before weighing?

- Press the Start key again - there should not be any load on the scale.

### ... - - - - appears instead of 0.00 before weighing?

- Press the Start key again - there should not be any load on the scale.

### ... one segment is illuminated constantly or not at all?

- The relevant segment has a fault. Inform Servicing.

### ... appears in the display?

- Battery voltage is running low. Change the batteries in the next few days.

### ... bAtt appears in the display?

- Batteries are empty. Put in new batteries.

### ... bAtt appears in the display while the seca solar panel (option) is connected?

- Not enough sun light. Angle the solar panel towards the sun.
- Disconnect the solar panel. The scale automatically switches to battery operation..

### ... 5tOP appears in the display?

- Maximum load has been exceeded.

### ... the display flashes?

- Take the load off the scale and start the scale again. Wait until 0.00 appears and then weigh again.

### ... the display Er and a number appear?

- Start the scale again after it switches off automatically. The scale will then work normally again.

If this is not the case, disconnect the power supply by briefly switching off the scale at the ON-OFF switch. If this measure is equally unsuccessful, inform the Service department.

## 7. Servicing


---

Your seca scale leaves the factory with an accuracy of  $\pm 0.15\%$ / $\pm 100\text{ g}$ / $\pm 0.22\text{ lbs}$ . To ensure that this degree of accuracy is maintained, the product must be carefully set up and regularly serviced. Depending on frequency of use, we recommend servicing at intervals of 3 to 5 years. Please note that a qualified servicing agent should be used for this. If you are in doubt, contact seca Service for further assistance.

### Changing the batteries

You need 6 type AA 1.5 V batteries.

- Carefully turn over the scale so that the base of the scale is accessible.
- Press the closure of the battery compartment in the direction of the cover itself and open the battery compartment.
- Remove the used batteries and insert new batteries.

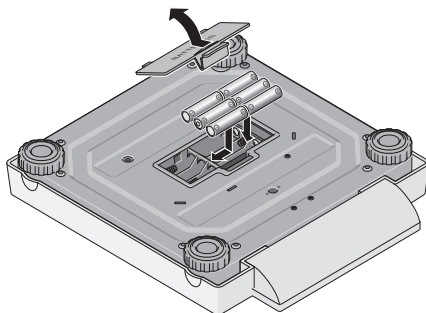### Check that the polarity is correct.

- Close the cover.
- Then turn the scale back up the right way.

## 8. Technical data

---

**Maximum capacity** 200 kg/440 lbs

### Graduations

|                    |         |
|--------------------|---------|
| 0 to 150 kg        | 50 g    |
| 150 kg to 200 kg   | 100 g   |
| 0 to 330 lbs       | 0.1 lbs |
| 330 lbs to 440 lbs | 0.2 lbs |

### Accuracy

|                    |                       |
|--------------------|-----------------------|
| 0 to 50 kg         | $\pm 100\text{ g}$    |
| 50 kg to 200 kg    | $\pm 0.15\%$          |
| 0 to 110 lbs       | $\pm 0.22\text{ lbs}$ |
| 110 lbs to 440 lbs | $\pm 0.15\%$          |

### EN 60 601-1

medical electrical equipment type B

### Size of digits

20 mm

### Dimensions

|         |        |
|---------|--------|
| Width:  | 321 mm |
| Height: | 60 mm  |
| Depth:  | 360 mm |

### Weight

approx. 4.0 kg/8.8 lbs

**Temperature range**

+10 °C to +40 °C

**Power supply**

6 type AA 1.5 V batteries  
mains unit (option)  
solar panel (option)

## 9. Accessories

---

**seca 421** carry case

Order no. 421-00-00-009

### Model-dependent mains unit

Plug-in mains unit, Euro:

Order no. 68-32-10-252

230 V~ / 50 Hz / 12 V= / 150 mA

Switched-mode mains unit:

Order no. 68-32-10-265

100-240 V~ / 50-60 Hz / 12 V= / 0.5 A

### Notice!

Use only seca original plug-in mains units with a controlled 12 V output voltage. Commercially-available mains units may deliver a higher voltage than they quote and this leads to damage to the scale.

### Mobile power supply

**seca 411** solar panel

Order no. 68-60-00-201

## 10. Disposal

---

### Device

Do not dispose of the device in domestic waste. The device must be disposed of properly as electronic waste. Follow the national regulations which apply in your case. For further information, contact our service department at: [service@seca.com](mailto:service@seca.com)

seca helps the environment. We are concerned to save natural resources. We therefore strive to save packaging mate-

rial where meaningful. What is left over can conveniently be disposed of on site via the recycling system.

### Batteries

Do not throw used batteries away in domestic waste. Dispose of batteries at collection points in the vicinity. When buying new batteries, select those low in harmful substances and containing no mercury (Hg), cadmium (Cd) or lead (Pb).

# 11. Warranty

---

A two-year warranty from date of delivery applies to defects attributable to poor materials or workmanship. All moveable parts - batteries, cables, mains units, rechargeable batteries etc. - are excluded. Defects which come under warranty will be made good for the customer at no charge on production of the receipt. No further claims can be entertained. The costs of transport in both directions will be borne by the customer should the equipment be located anywhere other than the customer's premises. In the event of transport damage, claims under warranty can be

honoured only if the complete original packaging was used for any transport and the scale secured and attached in that packaging just as it was when originally packed. All the packaging should therefore be retained.

A claim under warranty will not be honoured if the equipment is opened by persons not expressly authorised by seca to do so.

We would ask our customers abroad to contact their local sales agent in the event of a warranty matter.

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