VERMEIREN

286E

INSTRUCTION MANUAL
MODE D'EMPLOI
GEBRUIKSAANWIJZING
GEBRAUCHSANWEISUNG
ISTRUZIONI PER L'USO
MANUAL DE INSTRUCCIONES
INSTRUKCJA OBSŁUGI

Instructions to the specialist dealer

This instruction manual is part and parcel of the product and must accompany every product sold.

Version: A, December 2012

All rights reserved, including translation.

No part of this manual may be reproduced in any form what so ever (print, photocopy, microfilm or any other process) without written permission of the publisher, or processed, duplicated or distributed by using electronic systems.

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© N.V. Vermeiren N.V. 2012

EN Contents

1. Technical details	2
2. Components	2
3. Explanation of symbols	2
4. Check after delivery	2
5. Intended use	2
6. Fold and unfold the rollator	3
7. Assembly	3
8. Pushing handles	3
9. Brake system (Fig. 1)	3
10. Basket	4
11. Safety instructions	4
12. Care	4
13. Disinfection	4
14. Repairs / Service / Inspection	4
15. Shipping and storage	5
16. Warranty	5
17. Disposal	5
18. Declaration of conformity	5

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4-wheels rollator 286E

1. Technical details

⚠ WARNING: Risk of unsafe settings - Use only the settings described in this manual.

Make	Vermeiren	
Address	Vermeirenplein 1/15, B-2920 Kalmthout	
Type	Rollator	
Model	286E	
Width	590 mm	
Length (unfolded)	644 mm	
Seatheight	605 mm	
Seatwidth	430 mm	
Weight 10 kg	with basket & tray 11 kg	
Adjustable pushhandles	780 - 970 mm	
Wheels	Ø 191,5 mm x 58 mm	
Brakes	Multifunctional brake	
Max. inclination	use on flat underground	
Maximum user weight	max. 120 kg	
Maximum weight of the basket	max. 5 kg	
We reserve the right to introduce technical changes. Measurement tolerance +/- 15 mm / 1,5 kg		

First of all we wish to thank you for the trust you placed in us by selecting a **VERMEIREN** product.


Before using the rollator, read the instruction manual carefully: it will familiarise you with this product.

You will also be given hints about the care and service of your rollator. Please remember that observance of our hints would keep your rollator in an excellent condition of preservation and it will function perfectly even after years of use.

If you have any further questions, please consult your specialist dealer.

To find a service facility or specialist dealer near you, contact the nearest Vermeiren facility. A list of Vermeiren facilities can be found on the last page.

2. Components

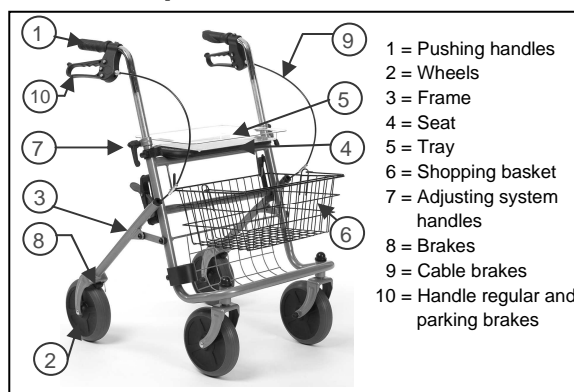3. Explanation of symbols

Maximum mass

Indoor and outdoor use

CE conformity

Read the instruction manual before use

Observe the safety instructions

4. Check after delivery

Unpack your product and verify if the delivery is complete. Following items should be included:

- rollator frame with bench, 4 wheels (mounted)
- height adjustable pushing handles with brakes
- shopping basket
- plastic tray
- manual

Verify your product for transport damage. If you find any damages after delivery proceed as follows:

- contact the transporter
- have a list made of any problems
- contact your supplier

5. Intended use

⚠ WARNING: Risk of falling - Do not use the rollator if you have weak arm function or poor equilibrium.

The rollator will compensate the lack of adequate support by the legs by leaning on, or moving with, the rollator with usage of the upper limbs or chest. The rollator will provide a better stability, support and safety during walking.



The rollator can be used indoor and outdoor. However, it cannot be used by persons with a visible weakened hand and/or arm function and also not by persons with a balance problem.

The product is not meant for transport of people and/or goods. The seating bench can only be used when the parking brake is locked.

6. Fold and Unfold the rollator

⚠ WARNING: Risk of clamping - Keep fingers, buckles and clothes away from the folding mechanism.

Use the lever on the right side of the upper frame to fold the rollator. After releasing the lever pull up the tube under the seat. For unfolding the rollator do the steps reversed.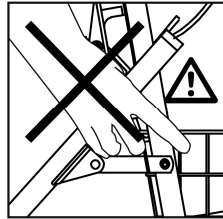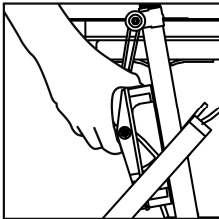

7. Assembly

- Slide the pushing handles in the front frame.
- If needed, adjust the cables connected to the brakes.
- Adjust the pushing handles to the desired height (see further).


8. Pushing handles

- ⚠ CAUTION: Risk of falling - Firmly fix the handles before use of the rollator.**
- ⚠ CAUTION: Risk of falling - Do not exceed the maximum adjustment, your rollator will not be stable anymore.**

- ⚠ CAUTION: Use the pushing handles only parallel to the walking direction of your rollator.**

The pushing handles can be adjusted in height. Unlock/release the lever ① on the side so that you can adjust the height of the pushing handles. When standing straight, the handles should be as high as your wrists. Leave your arms hanging loose. Lock the handles again firmly. The adjustment minimum and maximum can be seen on the marks on the inner tubes which slide into the frame. When you exceed the maximum adjustment, your rollator will not be stable anymore.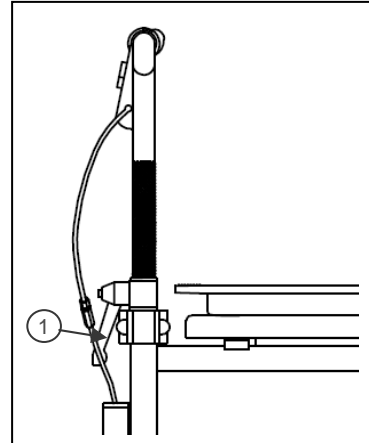


9. Brake system (Fig. 1)

- ⚠ WARNING: Good operation of the brakes is influenced by wear and contamination of the tires (water, oil, mud, ...) - Check the condition of the tires before each use.**
- ⚠ WARNING: The brakes can wear - Check the operation of the brakes before each use.**

The rollator has been equipped with a multifunctional brake system. The brake handles underneath the pushing handles are regular brakes as well as parking brakes. For the usage of a short brake (regular brakes), pull the brake handle upwards (toward the pushing handle). For the usage of a long brake (parking brake), push the brake handle downwards till you hear a locking sound of the brake. The wheels (use both parking brakes left and right simultaneously) are now blocked until you pull the parking brakes upwards again.

Fig. 1 – Pushing handle / brake

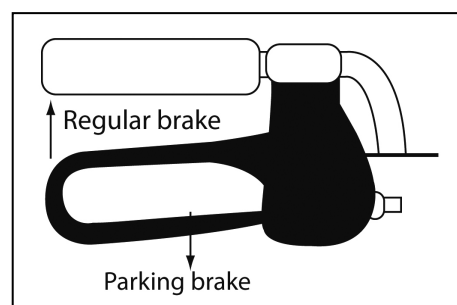10. Basket

⚠ WARNING: Risk of falling, tipping over - Do not overload the shopping basket.

The shopping basket is meant to be mounted on the front side. Two hooks will mount the shopping basket to the rollator frame.

The maximum carrying weight from the shopping basket is 5 kg.

11. Safety instructions

- Ensure there are no stones or any other objects blocking the regular usage of the wheels of the rollator.
- Ensure the cables or the brakes are not blocked or distorted.
- Ensure the brakes or the rollator are always in parking-mode before using your rollator. When you are standing comfortable and stable, you may unlock the parking brakes.
- Before every usage, check if the locking handle for your pushing handles is firmly locked.
- Verify if the safety against unwanted folding/unfolding is functioning properly.
- Only proceed moving with your rollator when you are standing comfortably and when both your hands are resting on the pushing handles.
- Do not hold the pushing handles with wet hands. If you do so, you can lose your grip and lose your balance.
- When you detect that your body is getting tired, lock the parking brakes of your rollator and have a seat on the bench.
- **The maximum users weight for your rollator is 120 kg** and can only be used on a flat underground.
- Do not use your rollator on a staircase or bumps higher than 40 mm.
- When using the rollator outdoors, find yourself flat and stable areas to use your rollator. Avoid kerbs and/or deep holes.
- Do not proceed with your rollator into traffic. Always stay on the pavement.

- Avoid bumpy or sandy roads. Otherwise you might lose control over the stability of your rollator.
- Avoid roads with a very steep inclination.
- Do not use your rollator to transport people or items.
- Risk of injury - Surfaces can assume the environment temperatures.
- Take note of the instructions for care and service. The manufacturer is not liable for damage caused by improper servicing/care.

12. Care

Use only a soapy solution for cleaning. The cleaning liquid should in any case have a pH value of at least 6. Do not use a cleaning agent which contains solvents. Observe the instructions of the cleaning liquid used. Use a damp cloth and lukewarm water for cleaning. Avoid soaking. Do not use metal brushes or any other sharp-edged or pointed cleaning utensils. In the case of damage to the varnish or the woodwork, consult your specialist dealer. Regular care would ensure that your side table is preserved in a perfectly functional condition.

13. Disinfection

For sterilisation use only disinfectants suitable for treating varnished metal and wood surfaces. Observe the instructions of the cleaning liquid used. Use only proved processes (for scrubbing disinfection) and disinfectants included in the list of the Robert Koch Institute (information can be obtained from: www.rki.de). Disinfections may only be undertaken by trained persons. Consult your specialist dealer.

14. Repairs / Service / Inspection

Lifetime of the rollator is influenced by its use, storage, regular maintenance, servicing and cleaning.

You should use only genuine VERMEIREN replacement parts. Repairs and renovations may only be undertaken by trained persons. Make use of the services offered by your specialist dealer. He would gladly assist you further in regard to service and repairs.

Before you resume use of the rollator, it should be serviced according to the following criteria:

- Completeness
- Adjustment functions

- Check the casters (stability, tracking, fixing screws)
- Setting of the adjustment levers (condition, thread, secure locking)
- Frames (deformation, stability, connections)
- Trays (undamaged surfaces, splinters, tears)
- **Damaged varnish (remove possible rust and repaint)**

Every 8 weeks:

- Lubricating (moving parts shall be greased)

Every 6 months or for every new user, it should be serviced according to the following criteria:

- General review
- Disinfection

15. Shipping and storage

The shipping and storage of the rollator shall be according following instructions:

- Store in a dry place (between +5°C and +41°C).
- The relative humidity of the air should be between 30% and 70%.
- Provide sufficient covering or packaging to protect the rollator from rust and foreign bodies (e.g. salt water, sea air, sand, dust).
- The rollator must be stored without being subjected to strains. (Don't put too heavy parts on the rollator, not clamping between something, ...)

16. Warranty

Excerpt from the "General Business Conditions".

(...)

5. The guarantee period for warranty claims is 24 months.

(...)

The warranty excludes damage arising from structural changes to our products, insufficient maintenance, defective or improper handling or storage or the use of non-original parts.

Likewise, the warranty excludes parts or working parts subject to natural wear and tear.

17. Disposal

When disposing of the rollator, contact your local disposal centre or return the product to your specialist dealer who, after submitting it to a hygienic procedure, will be able to send it back to the manufacturer who will dispose of and recycle it correctly, separating it into its component materials.

Packaging materials can be taken to disposal or recycling centers or to your specialist dealer.

18. Declaration of conformity

The manufacturer or his authorized representative :

N.V. VERMEIREN N.V

Address :

Vermeirenplein 1/15
2920 Kalmthout
Belgium

declares under his sole responsibility that the CE marked medical devices :

Productgroup: Rollators
Brand: Vermeiren
Type: 286E

have been classified as class I, according to annex IX MDD 93/42/EEC, rule 1,

and is manufactured in full conformity with the European instructions below - including the latest modifications - and with the national law, that organizes this directions :

Medical devices directive MDD 93/42/EEC: 2007

and is in conformity with the relevant European harmonized standards:

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ENGLISH

CONTRACTUAL WARRANTY

We offer 5 years of warranty on standard wheelchairs, lightweight wheelchairs 4 years. Electronic wheelchairs, tricycles, beds and other products: 2 years (batteries 6 months) and multiposition wheelchairs 3 years. This warranty is limited to the replacement of defective or spare parts.

APPLICATION CONDITIONS

In order to claim this warranty, part "B" of this card has to be given to your official Vermeiren dealer. The warranty is only valid when parts are replaced by Vermeiren in Belgium.

EXCEPTIONS

This warranty is not valid in case of:

- damage due to incorrect usage of the wheelchair,
- damage during transport,
- involvement in an accident,
- a dismount, modification or repair carried outside of our company and/or official Vermeiren dealership,
- normal wear of the wheelchair,
- non-return of the warranty card

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Vermeiren Suisse S.A.
Vermeiren Nederland B.V.